(DE)	Infrarotlampe		(RU)	Прибор инфракрасного излучения	
	<i>Gebrauchsanweisung</i>	2		<i>Инструкция по применению</i>	30
(EN)	Infrared lamp		(PL)	Lampa na podczerwień	
	<i>Instruction for use</i>	9		<i>Instrukcja obsługi</i>	37
(FR)	Lampe à infrarouge		(NL)	Infraroodlamp	
	<i>Mode d'emploi</i>	16		<i>Gebruiksaanwijzing</i>	44
(IT)	Lampada a raggi infrarossi				
	<i>Istruzioni per l'uso</i>	23			

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Content

1. Included in delivery.....	9	8. Changing the infrared lamp	12
2. About the infrared lamp	9	9. Cleaning and maintenance	13
3. Signs and symbols	9	10. Disposal	13
4. Intended use	10	11. Protective eyewear	13
5. Warnings and safety notes	10	12. Technical specifications	14
6. Initial use	11	13. Warranty / service	15
7. Usage	12		

1. Included in delivery

Check that the device packaging has not been tampered with and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1 x Infrared lamp with protective grille
- 1 x Protective eyewear
- 1 x These operating instructions

2. About the infrared lamp

This infrared lamp is only intended for radiating the human body.

Infrared radiation causes warmth to be transported to the human body. The blood supply in the radiated skin is increased and the metabolic turnover in the field of temperature enhanced. The mode of action of the infrared light encourages the body to heal; healing processes are thus assisted in a specific area.

Infrared light can be used as a parallel treatment in diseases of the ear, nose and throat and to assist in facial and beauty care, especially in case of impure skin.

However, first ask your family doctor whether application in the individual case is advisable from a medical standpoint.

Note: Warranty restriction

The infrared bulb included – as with all light sources – is not subject to the warranty.

3. Signs and symbols

The following symbols appear in these instructions or on the device.

	Observe the instructions Read the instructions before starting work and/or operating devices or machines		Serial number
	Protect from moisture		CE labelling This product satisfies the requirements of the applicable European and national directives.
	Protection class II device The device is double-protected and thus corresponds to protection class 2		Product information Note on important information

	Caution: hot surface		Manufacturer
	Warning Warning instruction indicating a risk of injury or damage to health		Disposal in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).
	Important Safety note indicating possible damage to the device/accessory		Distance between infrared heat lamp and irradiated body part
	Permissible storage temperature and humidity		Separate the packaging elements and dispose of them in accordance with local regulations.
	Permissible operating temperature and humidity		Marking to identify the packaging material. A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard
	Separate the product and packaging elements and dispose of them in accordance with local regulations.		

4. Intended use

The unit is intended for directing infrared radiation at the human body. Any other use is regarded as improper. This unit is not designed for commercial use, but for use in the private home!

5. Warnings and safety notes

Safety notes

Warning

- This infrared lamp is only intended for radiating the human body.
- Always use the protective eyewear supplied during usage.
- Always use the infrared lamp with the protective grille in place.
- Never remove the protective grille during use. Only remove the protective grille when changing the infrared lamp.
- Before use, ensure that all packaging materials are removed and that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact your dealer or the customer service address provided.
- Always ensure that the appliance is on a secure and even surface.
- The lamp housing and the infrared lamp become very hot during operation. There is a risk of burning if these are touched. Always let the lamp cool down first before touching it.
- Inflammable objects must be kept at a distance of at least 1.0 m from the infrared lamp. Do not hang the appliance on the wall or from the ceiling.
- Connect the unit only to the mains voltage listed on the type plate.
- Do not dip the unit into water and do not use it in wet rooms.
- This device is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety or they are given instructions on how to use the device.
- Do not use on babies and small children.
- Children should be supervised to ensure that they do not use the device as a toy.
- This device is not intended for use on changing units, cots or similar.
- Do not use it on persons sensitive to heat.

The sensitivity to heat can be reduced or increased in the following cases:

 - diabetic patients
 - persons with sleepiness, dementia or difficulties concentrating
 - persons with illness-related skin changes or persons with scarred skin areas in the application area
 - persons with allergies
 - children and older persons

- after consuming medications or alcohol
- Do not operate the device if you are wearing an implant, particularly if it is located in the area of the body to be treated. If you have any doubts, consult your doctor.
- If you are pregnant, do not operate the device near your abdomen, in order to prevent this area from overheating. If you have any doubts, consult your doctor.
- In case of acutely inflammatory processes, radiation should only be carried out after consulting a doctor.
- Always limit the period of use and check the reaction of the skin.
- Medications, cosmetics or food can lead to a over-sensitive or allergic reaction of the skin under certain conditions. Radiation must be ended immediately in this case.
- An appropriate distance between the infrared lamp and the desired treatment area must be maintained during treatment so that no skin irritation, such as tingling, itching, burning or too high heat sensitivity occurs. Stop the treatment if this occurs. Alternatively, increase the radiation distance and/or reduce the radiation duration.
- This device is not intended for radiation from above.
- The minimum distances described in these instructions for use must be adhered to.
- If the appliance is operated continuously, caution and attention are advisable - never fall asleep during radiation!
- Prolonged use can cause burns to the skin.
- Do not touch the infrared lamp or unscrew it when it is connected to the mains.
- Children do not understand the risks associated with electrical appliances. Make sure the appliance is not used by children without appropriate supervision.
- The appliance may be used only under supervision.
- Keep children away from packaging materials (risk of suffocation).
- Do not cover up or pack away the unit while it is warm.
- Always remove the mains plug and let the unit cool off before you touch it.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- Protect the unit from strong impacts.
- Do not pull the mains plug from the socket by the cord.
- In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply.
- If the power cord of the device is damaged, it must be replaced by the manufacturer or an authorised customer service technician in order to avoid hazard.
- Disconnection from the mains supply is only guaranteed if the mains plug has been removed from the socket.
- There is a risk of strangulation for babies and children who become trapped in the mains cable.

General notes

Important

- Product is not suitable for lighting rooms in private homes.
- This unit is not intended for commercial or clinical use, but only for individual, private household use!
- If you have health concerns of any kind, contact your general practitioner!
- Before you use the unit for the first time, remove all packaging materials.
- Lighting is excluded from the warranty.

Repairs

Important

- Do not open the device. Please do not attempt to repair the unit yourself. This could result in serious injuries. Failure to observe this regulation shall void the warranty.
- If you need to have the unit repaired, contact customer service or an authorised dealer.

6. Initial use

Setup

Remove the unit from the packaging. Remove all packaging material before using the appliance. Before use, check whether the infrared lamp is properly screwed in. Before use, check that the protective grille is securely in place. Place the device on flat surfaces. The appliance has an adjustable lampshade. Set the angle of the fluorescent screen in accordance with your individual requirements.

Mains connection

Connect the unit only to the mains voltage listed on the type plate.

**Note:**

- Ensure that there is a socket near the place where the lamp will stand.
- Lay the mains cable in such a way that no one can trip over it.

7. Usage

1	Switching on the lamp ► Push the plug completely into the mains socket.
2	Enjoying the light The recommended duration of treatment is maximum 12 minutes. We recommend selecting a shorter radiation period to start with. However, the skin of some people may show an over-sensitive (e.g. with extreme redness, blistering, itching, heavy sweating) or allergic reaction to heat radiation, even when the lamp is used correctly. If symptoms are not relieved after 6 to 8 applications, stop using the device and consult your doctor.
3	Important instructions The distance between the infrared lamp and the radiated part of the body should not be less than the following: at least 24 in (60 cm) Always use the protective eyewear supplied during usage. The protective eyewear corresponds to the requirements of EN 171:2002 and serves to protect the eyes against infrared radiation. CAUTION: Never look directly into the infrared light without protective eyewear. Basically, examine the radiated section of the body regularly. Discontinue radiation immediately if any signs of an over-sensitive or allergic reaction appear and consult a physician.
4	Switching off the lamp ► Pull the mains plug from the socket. CAUTION! The lamp is hot after being used. Let the lamp cool off first long enough before you put it away and/or pack it up! Please note that the mains cable must not be wound round the infrared lamp either for storage or when in use.

8. Changing the infrared lamp

The device must always be switched off, disconnected from the mains and allowed to cool down before cleaning and before changing the infrared lamp. The protective grille must also be removed. Please note that only a infrared lamp of the original type can be used as a replacement.

The procedure for replacing the infrared lamp is as follows:

1. To remove the protective grille, unscrew the screw on the side using a screw-driver.	1	2	3
2. Carefully pull the protective grille off the device.	4	5	6
3. Unscrew the old infrared lamp from the device.			
4. Fully screw the new infrared lamp into the device.			
5. Place the protective grille back on the device.			
6. Tighten the screw on the side of the protective grille using a screwdriver.			
7. Check that the protective grille is firmly attached to the device.			

Appliance:	Replacement infrared lamp (type):	Specifications:	Item number:
SIL 06	Infrared R95 (100 W)	230V ~ 50 Hz	614.51

Warning

- Do not touch the infrared lamp or unscrew it when it is connected to the mains.
- Unplug the device, to avoid the risk of electrocution.
- In the interest of environmental protection, do not dispose of the device in household waste.

9. Cleaning and maintenance

The unit should be cleaned occasionally.

Important

To avoid odours caused by fluff and dust, we recommend regularly cleaning the device as follows:

- The device must be switched off and allowed to cool and the mains plug pulled out before cleaning and changing the infrared heat lamp.
- Ensure that no water penetrates inside the unit!
- Do not wash the unit in a washing machine!
- Clean using a damp cloth, to which you can apply a little detergent if necessary.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- Do not use abrasive detergents and never immerse the unit in water.

10. Disposal

In the interest of protecting the environment, the unit must not be thrown out with the household waste at the end of its service life. Dispose of the unit at a suitable local collection or recycling point.

Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any queries, please contact the appropriate local authorities.

11. Protective eyewear

The product complies with the PPE regulation (EU) 2016/425 (reference: OJ L 81, 31/03/2016, p. 51-98), the harmonised standard EN 166:2001 (reference: OJ C 209, 15/06/2018, p. 18) and the standard EN 171:2002.

Warnings and safety notes

Read these instructions for use carefully! Non-observance of the following information may result in personal injury or material damage. Store these instructions for use and make them accessible to other users. Make sure you include these instructions for use when handing over the product to third parties.

WARNING

- Keep packaging material away from children. There is the risk of suffocation.
- Children must not play with the protective eyewear.
- Before using the protective eyewear, check it for signs of external damage (e.g. cracks or holes). If the protective eyewear is damaged, you must NOT continue using it.
- When wearing the protective eyewear, people with sensitive skin may have an allergic reaction to the materials of the eyewear that may come into contact with the skin.
- Make sure that the protective eyewear does not come into contact with grease, oils or alcohol (e.g. cleaning products or facial toners).
- Make sure that you do NOT reduce the distance between you and the infrared lamp whilst wearing the protective eyewear.
- Use the protective eyewear only in quiet environments, so that you are not disturbed during use.
- The protective eyewear is not suitable for use as eye protection against mechanical hazards (e.g. impacts).

Intended use

The protective eyewear is exclusively intended for protecting your eyes against infrared radiation while using an infrared lamp. The protective eyewear provides protection against infrared radiation only. The protective eyewear does NOT provide protection against other types of radiation. Use the protective eyewear only with Sanitas infrared lamps. Never use the protective eyewear with infrared lamps from other manufacturers.

Always use the protective eyewear directly over your eyes, not over another pair of glasses. The protective eyewear is intended for domestic/private use only, not for commercial use. It is free from pollutants as per the legal requirements.

Usage

1. Hold the protective eyewear to your eyes and pull the rubber band over your head. You can adjust the elastic band individually.
 2. Position the infrared lamp.
 3. Prepare the desired treatment distance.
 4. Switch on the infrared lamp.
 5. Sit down in preparation for the application.
- Replace the product after a period of use of 3 years.

Storage

If you do not plan to use the protective eyewear for an extended period of time, we recommend storing it in its original packaging in a dry environment, ensuring that it is not weighed down by other items.

Cleaning

Clean the protective eyewear from time to time. Use a slightly damp, soft cloth to clean it. Do not use any sharp or abrasive cleaning products.

Technical specifications

Model	SEP 02
Protection level	4-2,5 HAD 1 S - 166 S CE in accordance with EN 171:2002
Protective eyewear designation	4-2,5 HAD 1 S - 166 S CE The code for the lenses is comprised of the following: - Initial number for IR protection filter (1st part of code) - Protection level number (2nd part of code) - Manufacturer's code (3rd part of code) - 1 describes the optical class of the glasses (4th part of code) - S describes increased strength (5th part of code) - 166 is the normative basis (6th part of code) - CE mark (7th part of code)
Light transmission level	Max. 29,1% Min. 17,8%
Maximum average spectral transmittance value with infrared	780 – 1400 nm: 2.9% 780 – 2000 nm: 32.3%

Notified body:

ECS European Certification Service GmbH

Hüttfeldstraße 50

73430 Aalen, Germany

Identification number: 1883

EU declaration of conformity: <http://sanitas-online.de/web/de/landingpages/cedeclarationofconformity.php>

12. Technical specifications

Mains connection	220-230 V~ / 50-60 Hz
Power consumption	100 W
Dimensions (W x H x D)	122 x 205 x 180 mm
Max. Radiation intensity	680 W/m ² at 60 cm
Weight	approx. 331g
Operating conditions	Temperature: +5°C to +35°C Relative humidity: 15%-93%
Storage and transport	Temperature: -25°C to +70°C Relative humidity: ≤93%
Expected operating life of the device	7 years

The serial number is located on the device or in the battery compartment.

Subject to technical modifications. This device meets the requirements of the European Medical Devices Directive 93/42/EEC with amendments as well as the German Medical Devices Act.

13. Warranty / service

Further information on the warranty and warranty conditions can be found in the warranty leaflet supplied.

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